## Porcelain Etch & Silane

### **Description:**

Ultradent<sup>®</sup> Porcelain Etch is a buffered, gelled hydrofluoric acid to be judiciously used for intraoral or extraoral etching of porcelain.

Ultradent<sup>®</sup> Silane is a mono-component solution for application to porcelain following etching with Porcelain Etch.

### **Indications:**

Porcelain Etch is to be used for etching fractured dental porcelain and for etching porcelain veneers, crowns, or inlays in the office. Also use it for etching porcelain crowns before bonding orthodontic brackets.


**Important Note:** Hydrofluoric acid gives off a vapor that irritates respiratory passageways and on a chronic basis can damage them.

Ultradent Porcelain Etch is buffered, gelled, and dispensed in a way that minimizes these problems. Instructions must be followed to ensure removal of any harmful material. Silane is for bonding after etching in preparation for bonding resin. If proper precautions are followed, porcelain may be safely etched intraorally for bonding as follows:

### **Procedure:**

1. Freshen the porcelain surface to be repaired with a high speed diamond bur. If a previous repair is present, remove all old composite to a fresh porcelain surface.
2. Remove luer lock caps from syringes. For optimum control, hold syringes in the palm of the hand.
3. Isolate/protect adjacent tissues and restorations by applying a bead or layer of EtchArrest<sup>®</sup> acid barrier/neutralizer medium with the Black Micro<sup>®</sup> tip (Fig. 1). An alternative isolation is a rubber dam (DermaDam<sup>®</sup>) and rubber dam sealant (OraSeal<sup>®</sup>).
4. Twist an Inspiral<sup>®</sup> Brush tip securely onto syringe. The Inspiral Brush tip ensures controlled flow from the tip.
5. Before applying, dispense a small sample of etchant from tip onto a mixing pad to ensure evenness and to become familiarized with flow rate. If there is excess resistance to flow, DO NOT proceed. Attach a new tip and test again before using on patient. Dispose of removed tip immediately (according to government regulations) to avoid contact.
6. Express an even coating of Porcelain Etch onto prepared porcelain (Fig. 2) using the Inspiral Brush tip. Etch porcelain one minute (note Figs. 3 and 4 showing before and after SEMs).


7. Assistant should hold vacuum tip adjacent to the surface while the dentist dispenses the Porcelain Etch. This ensures removal of any irritating vapors while acid reacts on porcelain (one minute). To neutralize the Porcelain Etch, blend EtchArrest into the etchant until the amber color of the etch is no longer identifiable.
8. Remove neutralized acid by placing the vacuum tip next to the surface and then rinsing with water spray. Rinse thoroughly and dry. Porcelain should have a frosted appearance. Repeat etching if not.
9. Apply Silane with a Mini Brush tip. Let evaporate for one minute. If not completely dry after one minute, air dry with oil-free air.
10. Place bonding resin according to product instructions. Important Note: Testing shows highest bonding values when etched porcelain is silanated with Ultradent Silane followed by PQ1® single syringe bonding agent in the following manner: Apply Silane to porcelain using a Mini Brush tip. Gently brush the Silane into the porcelain for one minute using small strokes. Air dry and proceed with application of PQ1.

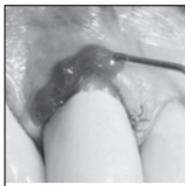
*Fig. 1 Protective bead of EtchArrest is placed on soft tissues adjacent to the tooth surfaces.*

*Fig. 2 Carefully express an even coating of Ultradent Porcelain Etch to prepared porcelain.*

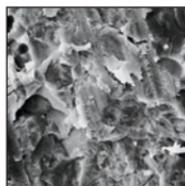
*Fig. 3 SEM of diamond-ground porcelain.*

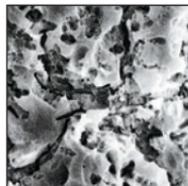*Fig. 4 Note increased surface area and undercuts afforded by etch with Porcelain Etch for one minute.*

### **Precautions:**

1. Operator, assistant, and patient must wear protective eyewear.
2. A clean, dry, well-etched surface is mandatory for a quality bond! We recommend using Aqua✓™ daily to keep contaminated air out of the three-way syringe.
3. Replace luer lock caps on syringes when not in use. Tips are disposable.
4. We recommend Ultradent® Syringe Covers sealed with the Ultradent® Impulse Sealer for a protective barrier to prevent cross-contamination.

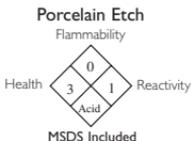**HAZARD RATING**

- 4 = Severe
- 3 = Serious
- 2 = Moderate
- 1 = Slight
- 0 = Minimal

**Shelf Life: 36 months. Store at room temperature.**

Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers.

The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

**Caution: Contains Hydrofluoric Acid**

U.S. federal law restricts the sale of this device by or on the order of a dentist.  
Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free **1-800-552-5512**.

Outside U.S. call (801) 572-4200  
or visit [www.ultradent.com](http://www.ultradent.com).

© Copyright 2005 Ultradent Products, Inc. All Rights Reserved.

U. S. Patent Nos.: 4,997,371; 5,246,371; 5,269,684; 5,534,562; 5,700,148; 5,708,052; and 5,722,833.

Other U.S. and International Patents Granted and Pending.

**Key:** **Legende:** **Explication:** **Erklärung der** **Packungssymbole:** **Teckenförklaring:** **Use by Date** **Verwendbar bis** **Houlder avant** **A utbildbarheidsdatum** **Usare entro la data indicata** **Verwendbar bis** **Clave:** **Legenda:** **Teckenförklaring:** **Use by Date** **Verwendbar bis** **Houlder avant** **A utbildbarheidsdatum** **Usare entro la data indicata** **Merkkien selitykset:** **Kλειδί:** **Tegnforklaring:** **Merkkien selitykset:** **Κλειδί:** **Обозначения:** **Oznaczenia na opakowaniach:** **Anahtar:** **Обозначения:** **Oznaczenia na opakowaniach:** **Anahtar:** **要點** **ໜ້າສຳຄັນ** **主要なシンボル:**

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