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*wo erforderlich

DESCRIPTION OF COMPONENTS

- 1.surgical binocular loupes
- 2.side shields*
- 3.adjustable headband
- 4.microfiber cleaning cloth
- 5.screwdriver
- 6.anti-fog cleaner
- 7.instruction booklet

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To avoid damaging the eyes, do not look directly at the sun or sources of strong light when wearing these binoculars.

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Disposal: Refer to the instructions as reported on "MAGNIFICATION SYSTEMS MAINTENANCE AND WARRANTY" – Prior to dispose of, it is mandatory to disinfect the article as a whole.

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Warranty: Refer to the instructions as reported on "MAGNIFICATION SYSTEMS MAINTENANCE AND WARRANTY"

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Environmental and storage conditions: We suggest to use these articles in an environment which temperature ranges between 10°C and 40°C – to store the same articles when not in use it is advisable not to exceed the following range of temperatures -10°C and +50°C. It is wise to store these articles away from heat sources and in a dry environment free of organic vapors.

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The table below shows the technical features of the various models.

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MODEL	CODE	MAGNIFICATION	WORKING DISTANCE
GALILEAN TTL	TGU20	2,0x PRO™	from 300 to 500
GALILEAN TTL	TGU25	2,5x PRO™	from 300 to 500
GALILEAN TTL	TGU30	3,0x PRO™	from 300 to 500
GALILEAN TTL	TGU35	3,5x	from 300 to 500
PRISMATIC TTL	TKU30	3,0x	from 300 to 500
PRISMATIC TTL	TKU35	3,5x	from 300 to 500
PRISMATIC TTL	TKU45	4,5x	from 300 to 500

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APPLICATIONS - Custom-made surgical eyewear with magnification can be used to see tiny details during surgery while maintaining an optimal working position. The telescopes, inside the basic spectacle lenses, are positioned according to the characteristics of each user (interocular distance, dioptric correction, etc.) to ensure maximum comfort during use. Binocular loupes are used in microsurgery, dental mechanics, aesthetic and vascular surgery, and heart surgery. They are not recommended for use in neurosurgery and other applications requiring very high image stability, which is not possible with devices mounted on the surgeon's head.

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HOW TO PLACE AND ADJUST THE HEADBAND - Loupes should be used with the headband fastened to keep them in position. Put on the headband and adjust the Velcro fastening to regulate the tension of the headband and get a comfortable fit.

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IF YOU SEE AS ABOVE - The loupe is too high: widen the adjustable nose pad.

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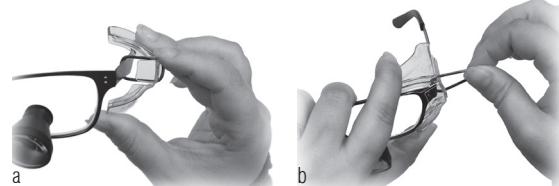
IF YOU SEE AS ABOVE - The loupe is moved toward right: widen the left part of the nose pad.

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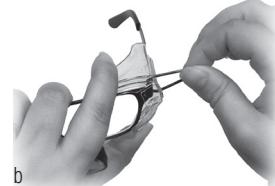
HOW TO MOUNT THE SIDESHIELDS - Insert left temple through large loop of black mounting band on the inside of the left sideshield (a). Hold frame and sideshield firmly while stretching the band outward and back to fit into the retaining grooves and over the rear tab (b). Repeat these steps for the right side.

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IF YOU SEE AS ABOVE - The loupe is too low: tighten the nose pad.

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IF YOU SEE AS ABOVE - The loupe is moved toward left: widen the right part of the nose pad.

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